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Representação na EU:
Advena Ltd.
Tower Business Centre,
2nd Flr., Tower Street,
Swatar, BKR 4013
Malta

QUATRON
Length adjustable multi needle
Instruction for Use
(Read all product labeling before use)


0344

Sterile, Single Use Only, Non-Pyrogenic, Non-Toxic

Shape/Structure and Principle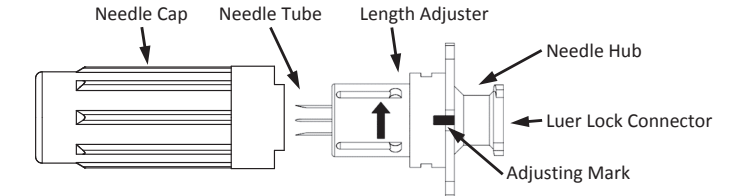

Materials to be exposed to the body
Needle Tube: Stainless Steel
Hub: Polycarbonate
Length Adjuster: Polystyrene

Intended Use

Single-use sterile needle intended to be used for medical purposes to inject fluids into intradermal tissue, subcutaneous tissue or muscle tissue.

Contraindication

QUATRON is single use only. Do not resterilize or reuse. Reuse of the device could increase the risk of infection. When needle tubes are completely covered by Length Adjuster, it should be regarded as after use.

Description

QUATRON consists of three components, stainless needle tubes, a length adjuster and a hub. The hub has female lure lock connector to fit to injection syringes. QUATRON is available in 31G and 34G.



Gauge	Outside Diameter (mm)	Length (mm)
31G	0.26	0 to 4 adjustable
34G	0.18	0 to 4 adjustable

Package

10 pieces/box

Directions for Use

- 1) Prepare the items below.
QUATRON, a sterile syringe, a sterile general needle
- 2) Attach the sterile general needle to the sterile syringe and draw up inject fluids in accordance with your institution's protocol.
- 3) Remove the general needle from the syringe in accordance with its instruction for use.
- 4) Connect the syringe to the Needle Hub of the QUATRON.
- 5) Pull the Needle Cap straight off the QUATRON.
- 6) Hold the flange of the Needle Hub and turn the Length Adjuster in the direction of the arrow marked on QUATRON for desired Length. Adjusting mark can be used as a guide to know the Length. One revolution of the length adjuster in the direction of the arrow provides 1mm shorter Length. The mark should be used only as a guide.
- 7) Holding the syringe with the needle pointing up, check the syringe for bubbles. If there are bubbles, gently tap the syringe with your finger to remove any visible bubbles and expel them from the syringe.
- 8) Insert the needles of the QUATRON in a direction perpendicular to the skin of the injection site and administer inject fluids in accordance with your institution's protocol.
- 9) Turn the length adjuster in the direction of the arrow marked on the QUATRON until it stops and the needle tip is completely covered. Then recap as necessary.

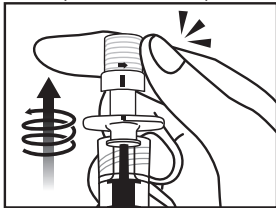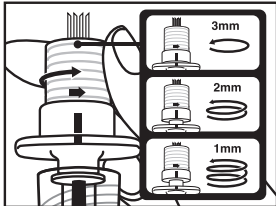- 10) Discard QUATRON immediately as infectious waste in accordance with your institution's protocol.

Important Cautions

- ASTI Corporation accepts no responsibility for accidents or defectives caused by misuse or abnormal use.
- Do not use this product when there is any breakage/defacement on package or abnormality in the product itself.
- Use the product immediately after unsealing the package.
- Do not touch the lure lock connector directly before use to avoid contamination.
- Do not touch the needle tubes directly while and after needle cap is removed to avoid needle stick injury or infection.
- Ensure the QUATRON fits syringes firmly and confirm there is no defect in the fitting, like leakage.
- Make sure to use luer lock syringe when infuse viscous fluids. Using luer slip syringe may cause detachment of QUATRON during injection.
- Do not use QUATRON to draw up inject fluids from vials or ampules. (This may cause impairment of the product)
- Select appropriate gauge and adjust Length according to the target administration area. Administering with short Length less than 1.5mm is likely to cause inject fluids leakage.
- Ensure the needles are inserted perpendicularly to the skin to avoid leakage of inject fluids.
- Hold the hub of the QUATRON during injection to avoid injury or leakage of inject fluids.
- Do not excessively rotate while operating the length adjuster or attaching to syringes.

Note

If there is any serious incident occurred in relation to the QUATRON, do report to ASTI and competent authority which your institution belongs to.

Storage Method and Expiration Date

- Keep the product in clean area, and avoid exposure to water, high temperature/humidity and direct sunlight.
- Expiration date is indicated on the individual package.

Manufactured by:
ASTI Corporation
1-5-1, Shinmiyakoda, Hamana-ku,
Hamamatsu, Shizuoka
431-2103, Japan
E-mail:medical@asti.co.jp

EU Representative:
Advena Ltd.
Tower Business Centre,
2nd Flr., Tower Street,
Swatar, BKR 4013
Malta

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- ASTI Corporation no asume ninguna responsabilidad por lo accidentes causados por uso incorrecto o anómalo.